IDK Extract[®]

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***Stool sample preparation system filled with extraction buffer
for use in ELISA***

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1. INTENDED USE

This stool sample preparation system *IDK Extract*[®] is intended for the sample preparation of stool samples to be used in the following Immundiagnostik ELISAs:

- Calprotectin (K 6927, K 6967)
- Pancreatic elastase (K 6915)
- Lactoferrin (K 6870)
- Hemoglobin (K 7816D, K 7836D)
- Hemo-haptoglobin complex (K 7817D, K 7837D)
- Anti-transglutaminase antibodies (K 9393)
- Anti-gliadin antibodies (K 9311)
- sIgA (K 8870, K 8880)
- α 1-antitrypsin (K 6750, K 6760)
- Albumin (K 6330)
- EDN (K 6811, K 6821)
- Lysozyme (K 6900, K 6901)
- b-Defensin 2 (K 6500)
- Bile acids (K 7878)

For *in vitro* diagnostic use only.

2. MATERIAL SUPPLIED

| Cat. No. | Label | Kit components | Quantity |
|----------|--------------------------|--|-----------|
| K 6999 | IDK Extract [®] | Stool sample tube filled with 1.5 ml <i>IDK Extract</i> [®] | 100 units |

3. STORAGE AND PREPARATION OF REAGENTS

- The **extraction buffer *IDK Extract*[®]** is stable until the expiry date when stored at **2–30°C**.

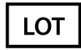
4. TECHNICAL HINTS

- Do not interchange different lot numbers of any kit component within the same assay.
- Reagents should not be used beyond the expiration date stated on kit label.
- Avoid foaming when mixing reagents.
- Do not mix plugs and caps from different reagents.
- The assay used should always be performed according to the enclosed manual.

5. GENERAL NOTES ON THE TEST AND TEST PROCEDURE

- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to Immundiagnostik AG along with a written complaint.

Used symbols:

| | | | |
|---|------------------------------------|---|-----------------------------------|
|  | Temperature limitation |  | Catalogue Number |
|  | In Vitro Diagnostic Medical Device |  | To be used with |
|  | Manufacturer |  | Contains sufficient for <n> tests |
|  | Lot number |  | Use by |
|  | Attention |  | Consult instructions for use |
|  | Consult specification data sheet | | |